

SURGICAL FACE MASK



FITTING INSTRUCTIONS

Sleeved metal nose band on top



Ensure that the mask is correctly configured before putting it on. The blue tinted fabric should be outward facing and if you feel for the small metal nose band you can ensure that it is on the top of the mask



Present the mask to the face in the above orientation.



Slip the elastic ear loops over your ears and ensure that your nose and mouth are fully covered.



The masks are pleated and you will need to pull the pleats open so that the mask fits securely under your chin.



Mould the metal strip around your face by pinching it against the bridge of your nose.

Should you want to tighten the mask you can do so by looping the elastic ear straps.

